

Results of tests with the resonance wave therapy device, Aquatone at FTO Mauz dental clinic no. 2.

After the acquisition of Aquaton unit at FTO MAUZ dental clinic No. 2, over 30 patients have been treated, mainly with the following diagnoses:

- Arthritis in the temporomandibular joint (dysfunction, pain, or incomplete opening of the mouth), 10 patients with positive results, best effects is achieved in combination with phonophoresis (8-10 sessions with mode 2 for 10 minutes, 1 time per day);
- The pain after root canal treatment of 14 patients, the Aquatone device demonstrated analgesic effects in combination with pulse therapy;
- The treatment of post-traumatic hematoma and post-extraction (5 patients) with Aquatone device demonstrated rapid resorption of the hematoma (after 5-6 sessions with mode 2 for 10 minutes per day).
- At maxillofacial fracture, the Aquatone device was used as anti-inflammatory aid, after 8-10 sessions, a decreased pain was noted as well as decreased swelling of the tissue. After 10 days of treatment with the device, the addition of calcium chloride is recommended.

The usage of the Aquatone device accelerates the rehabilitation process.